



Modifying Trim Lines of an OBSS Digitized Image

Changing Overall Width of the Cushions (Seat and Back)

1. Click the “Front” view icon located above the digitized image in order to view the cushion from one side or the other. (Fig.1)
2. Click on the “Edit Cushion” pull down menu and scroll down to “Modify Trim Points”; click on the “Modify Trim Points” option. You should now see yellow boxes located at strategic points on the green trim lines. (Fig.2)

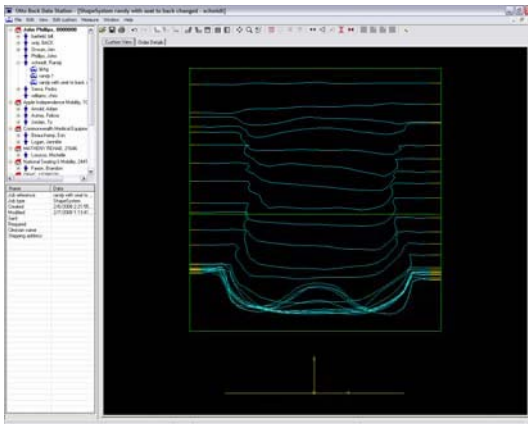


(Fig. 1)

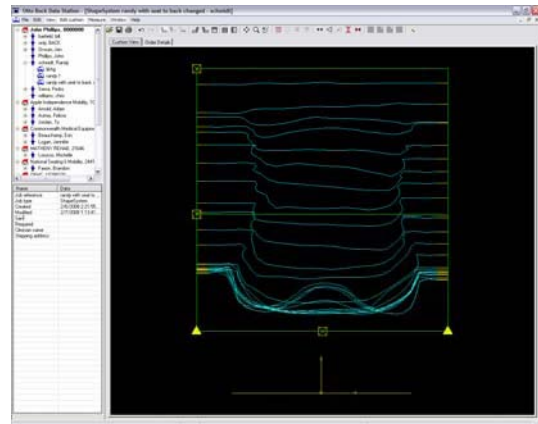


(Fig. 2)

3. Using your mouse cursor, left mouse click on the yellow triangle located on either side of the image. The yellow triangle will now change to a pink triangle. (Fig.3)
4. Place the mouse cursor at the desired location for your new width, and left mouse click once. The pink triangle will move to the location of your mouse cursor and relocate the green trim lines accordingly, and the pink triangle will change back to yellow. (Fig.4)

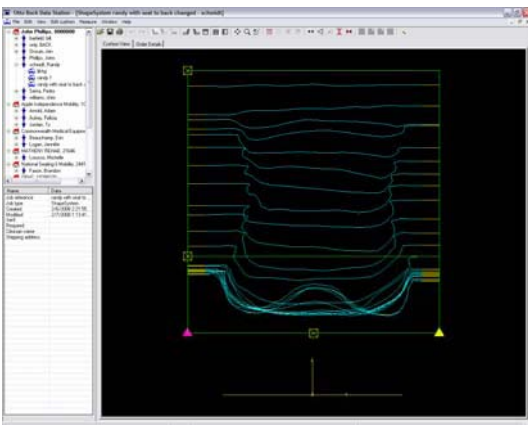


(Fig. 3)

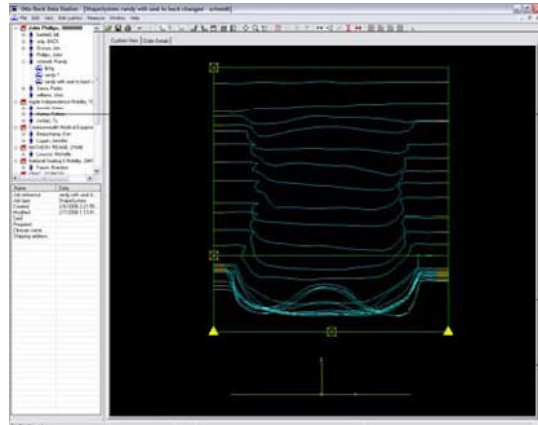

(Fig. 4)

5. Click on the “Edit Cushion” pull down menu and scroll down to “Modify Trim Points”; click on the “Modify Trim Points” option to turn the feature off and hide the yellow triangles.
6. Click on the “Edit Cushion” pull down menu and scroll down to “Cutting Box Dimensions”; click on the “Cutting Box Dimensions” option to view your new overall width. (Fig.5)

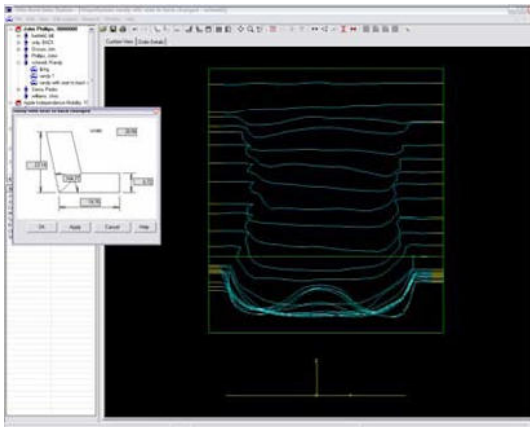

(Fig. 5)

7. For more modifications of the same measurement, repeat steps 1 through 6.

Note: This “Tech Tip” will work for a single cushion as well.

[Download](#) a copy of this tech tip or our other tips.