Otto Bock®

QUALITY FOR LIFE

8S4=*, 8S5=*, 8S6=*, 8S11=*, 8S12=*, 8S13=*, 8S20=*
MyoSkin Natural, MovoSkin Natural, PhysoSkin Natural

Ⓓ	2	Ⓟ	34
ⒼⒷ	5	ⓗ	38
Ⓕ	9	ⒸⓏ	41
Ⓡ	13	ⓇⓄ	45
Ⓔ	16	ⓉⓇ	48
Ⓟ	20	ⒼⓇ	52
Ⓝ	23	ⓇⓊⓈ	55
Ⓢ	27	ⒷⒼ	59
ⒹⓀ	31	Ⓢ	63

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Explanation of Symbols

English

NOTICE Warnings regarding possible technical damage.

INFORMATION Additional information on the fitting/use.

INFORMATION

Please read the Instructions for Use carefully. Please pay special attention to the listed safety instructions.

1 Description

1.1 Intended Use

The MyoSkin, MovoSkin and PhysoSkin Natural Series Gloves are intended **exclusively** for use with Otto Bock upper extremity prostheses.

1.2 Safety Instructions

Please make your patients aware of the following safety instructions:

NOTICE

Risk of damage caused by improper cleaning agents. In order to avoid damage to the material and the resulting permanent soiling, no solvents such as thinner, acetone, gasoline, etc. and no abrasive materials should be used for cleaning.

The Special Cleaner 640F12 should be used for cleaning stubborn dirt.

INFORMATION

The Natural Series gloves have a somewhat smoother surface. As a result, they may provide less grip than conventional PVC gloves during some activities (driving, sports, etc.).

2 Handling

2.1 Information for the orthopaedic technician

Please apply only a small amount of Procomfort Gel 633S2 to the inner hand as a donning assist. The gel creates a temporary sliding film which makes it easier to pull the glove over the inner hand. After a short time, the sliding film disappears and the glove fits properly.

NOTICE

Damage to the material. If materials which are not recommended are used as donning aids, damage to the material such as discolouration may result.

NOTICE

Functional limitations when silicone oil is used. Do not use silicone spray under any circumstances when putting on. The silicone film does not disappear and the glove cannot stabilize on the inner hand. Twisted fingers, a poor fit and reduced functionality of the prosthesis result.

By cautious warming of the glove before donning, you can increase its flexibility. The functionality of the prosthesis can be improved by carefully warming the palm side of the glove after putting it on and then slowly opening the hand. The glove is cooled down in the open state.

NOTICE

Damage caused by heat. The glove must not be warmed above 60 °C.

NOTICE

Damage through overstretching. The Natural Series gloves must not be stretched excessively (max. 50%) while warm.

The gloves are coloured by hand. Therefore, there may be slight differences between the same gloves.

INFORMATION

Colouring the MyoSkin, MovoSkin and PhysoSkin Natural Series gloves using the Otto Bock Colour Pens 635P15=2 (red) and 635P15=5 (blue) is not possible.

2.2 Cleaning and care information

INFORMATION

Please make your patients aware of the following information. Use the patient information 646D423 to do so.

The following information allows your patients to maintain the attractive and natural appearance of the prosthetic glove as long as possible:

In most cases daily hygiene (regular cleaning with soap and water) is sufficient to keep the glove clean.

Many stubborn stains can be removed with the Otto Bock Special Cleaner 640F12. Patients can obtain the Special Cleaner 640F12 and the matching Pump Sprayer 640F13 at medical supply stores.

To clean, apply the Special Cleaner 640F12 to the contaminated area using the pump sprayer. Before the cleaner dries, use a dry, clean, undyed cloth (cellulose paper, linen, etc.) to absorb the dissolved contaminant. Then rinse with water.

Avoid any contact with substances that strongly dye such as print colours, carbon copy paper, ballpoint pen ink etc. In order to reduce bleeding, colourful articles of clothing should be washed according to the manufacturer's recommendations before being worn for the first time.

Protect the prosthesis with a protective glove while carrying out household, garden or other activities that cause dirt and dust.

The fingernails can be painted with regular nail polish. To remove, use acetone-free (!) nail polish remover. Then the nail can be painted again.

3 Liability

The manufacturer assumes liability only if the user observes the product's specified fabrication and use instructions, adheres to its care instructions, and follows its service schedule. This product may only be used in combination with components that have been authorized by the manufacturer (please refer to the instructions for use and catalogs). The manufacturer does not assume liability for damage caused by component combinations that were not authorized by the manufacturer.

4 CE Conformity

The CE-Marking has been carried out in accordance with the requirements of the European Directive for Medical Products 93/42/EEC.

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